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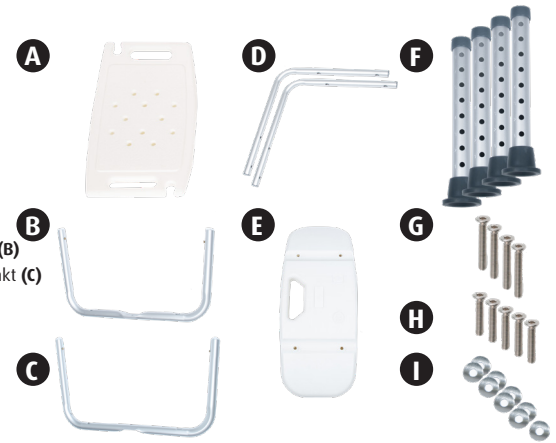
🇩🇪 Parts:

- Seat (A)
- Underframe with U-shaped centre (B)
- Underframe with spherical centre (C)
- 2x frame backrest (D)
- Backrest (E)
- 4x adjustable leg (F)
- 4x screws 58mm (G)
- 5x screws 38mm (H)
- 9x rings (I)

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🇩🇪 Place seat (A) bottom up on the floor. Place underframe with U-shaped centre (B) at an angle to the seat in the recess provided for that purpose recess. Then place the underframe with spherical centre (C) crosswise over the first underframe (B). As both bases contain a notch, the frames fit together neatly. Then first attach 1 screw 38 mm (H) with ring (I) in the centre.

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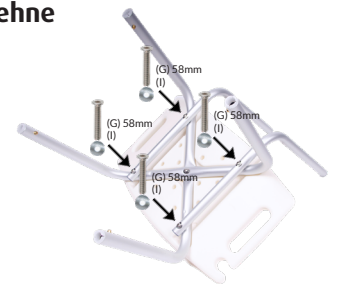
🇩🇪 Place everything you assembled in step 1 on a table or chair. Then place both backrest frames (D) on top of the base so that the holes in which the screws are to be fastened overlap. Then attach 4 screws 58mm (G) with rings (I) in the holes provided.

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
🇩🇪 Place backrest (E) and fasten it with 4 screws 38mm (H) and rings (I). Then place the 4 adjustable legs (F) on the frames. Pass the silver pin through the desired hole so that you reach the correct height. It is important that each pin is at the same height with all 4 legs.

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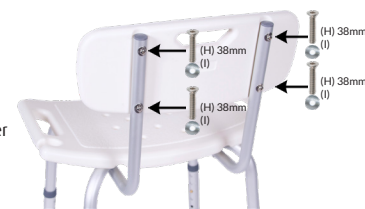🇩🇪 When the 4 legs are fixed at the correct height, the Dunimed shower stool is ready for use.

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